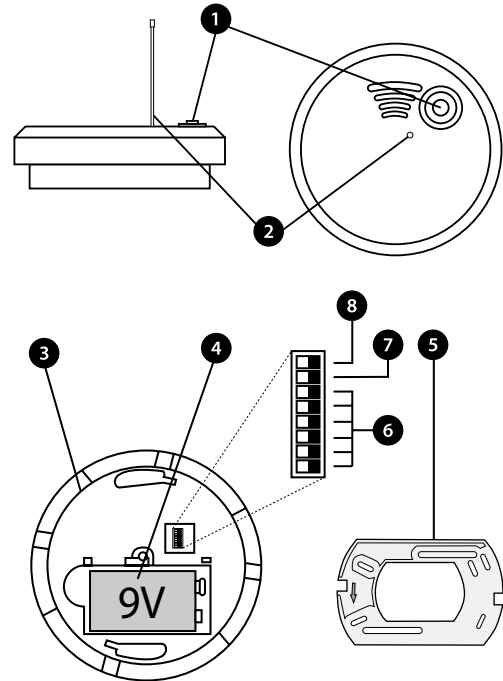
**DEUTSCH**

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## Appendix - Further information

### Connection

The Visit Smoke Alarm does not require any connections.

### Settings

No adjustments are required for normal use. The relevant descriptions are provided below, if you wish to change a setting for some reason.

### Radio key

In order to use several Visit Systems close to one another without interference, different Radio Keys can be set on the different systems. All Visit System units are supplied from the factory tuned to the same Radio Key, channel 0. This means that all Radio Key Switches on the transmitters are set to the OFF position.

- To alter the Radio Key, move the Radio Key Switches (6) to the desired positions. Note that switches (7) and (8) are not Radio Key Switches.
- The broadcast switch (7) has the function of switching on and off an additional safety function which makes the smoke alarm transmit to all receivers within its range regardless of the Radio Key that is set.

Please note: all Visit products within a system must be tuned to the same Radio Key in order to operate as a group. If the broadcast switch (7) is in the ON position, however, all Visit receivers are activated when a fire alarm signal is detected regardless of the Radio Key. If an incorrect Radio Key is set on the Smoke Alarm, neither the battery warning nor the Test Button will work with the Visit System's receivers. This function ensures that, when in Broadcast mode, you will not interfere with other Visit Systems when you test the smoke alarm or have a low battery.

### Signal pattern

A Signal Pattern is the name for the way in which a receiver in the Visit System indicates activation. Changing the transmitters' Signal Switch changes the Signal Pattern which the receivers display when the transmitter is activated.

The following signal patterns are available for the Visit Smoke Alarm:

Type	LED pattern	Sound	Vibration	Flash
Fire alarm	Red blinks constantly	Fire alarm constant	Long	Yes
Fire alarm low battery	Red blinks every five seconds	No	No	No
Fire alarm flat battery	Red blinks every five seconds	Fire alarm one short	One short	Yes

### Fire alarm system with Visit Guard Receiver

The Visit Smoke Alarm can be connected to a special fire alarm system in which a BE1465 Visit Guard Receiver controls all the smoke alarms that are registered on the system. The Guard Receiver Switch (8) is used for this function.

If no BE1465 Visit Guard Receiver is used in your system, the Guard Receiver Switch (8) is in the OFF position.

See the Guard Receiver User Manual for further information.

### Testing

It is easy to test the Visit Smoke Alarm. If the Smoke Alarm does not work as described below, you can check further under Troubleshooting/Troubleshooting guide.

### How to test

Test the Smoke Alarm at least once per month and preferably each week, e.g. during cleaning. Always test it immediately after any holidays or other extended periods of absence.

A receiver in the Visit System which is set to the same Radio Key as the Smoke Alarm is required to test the Smoke Alarm with the Test Button (1) or by blowing in smoke. Proceed as follows:

- Hold down the Test Button (1) or blow smoke into the Smoke Alarm. The Visit Smoke Alarm will transmit a radio signal after about five to ten seconds.
- The Visit receivers will then indicate the Smoke Alarm.

## Troubleshooting

You can carry out a number of checks yourself before sending a product for repair.

### Troubleshooting guide

Problem	Solution
Nothing happens when the transmitter is activated with the Test Button (1).	<ul style="list-style-type: none"> <li>Change the battery. Use a Duracell MN1604, Energizer 522 (alkaline) or Ultralife U9VL-J-P (lithium).</li> </ul>
The Visit Receivers occasionally indicate a fire alarm for no apparent reason.	<ul style="list-style-type: none"> <li>Change the battery. Use a Duracell MN1604, Energizer 522 (alkaline) or Ultralife U9VL-J-P (lithium).</li> <li>Change the Radio Key on all units in the system. For further information see Function/Radio Key.</li> </ul>
The Smoke Alarm can be activated with the test button (1) but the receivers are not responding.	<ul style="list-style-type: none"> <li>Check that the Smoke Alarm's antenna is straight and points to the floor.</li> <li>Check that the receivers are not placed too far away by moving them closer to the Smoke Alarm.</li> <li>Check the battery in the receiver.</li> <li>Check that all units in the Visit System are set to the same Radio Key. For further information see Appendix/Further information/Settings/Radio Key.</li> </ul>
The LED (1) blinks yellow or in a strange manner.	<ul style="list-style-type: none"> <li>Refer to the Guard Receiver User Manual if the Smoke Alarm is connected to a system with a BE1465 Visit Guard Receiver.</li> <li>If the Smoke Alarm is not connected to a system with a BE1465 Visit Guard Receiver, check that the Guard Receiver Switch (8) is in position 0.</li> </ul>
The receiver signals when no transmitter is activated.	<ul style="list-style-type: none"> <li>Change the Radio Key on all units in the relevant Visit System. There is probably another system nearby with the same Radio Key.</li> </ul>
The Smoke Alarm beeps and chirps in a different way from that indicated above for no reason.	<ul style="list-style-type: none"> <li>The Smoke Alarm is defective and needs to be sent for repair.</li> </ul>

## Technical information

### Power supply

Battery power 9 V battery: Duracell MN1604, Energizer 522 (alkaline) or Ultralife U9VL-J-P (lithium)

Operating time Duracell MN1604/Energizer 522:  
On the Visit System: approximately 3 years  
On the Fire Alarm System: approximately 2 years  
Ultralife U9VL-J-P:  
On the Visit System: approximately 6 years  
On the Fire Alarm System: approximately 4 years

Power consumption Active: 40 mA  
Idle position: < 10 µA

### Radio function

Radio frequency 868.3 MHz

RF radio Power Maximum transmission power: 5.6dBm

Number of Radio Keys 64 Radio Keys as standard. Special software can be used to increase these to 256 Radio Keys in increments of 64 per software purchase. Contact the nearest supplier for further information.

Coverage The normal coverage between a transmitter and receiver in the Visit System is approximately 200 metres with a clear line of sight. Coverage is reduced if walls and large objects screen off the signals. Any thick walls constructed of reinforced concrete will greatly affect coverage. The system may also be affected by radio transmitters such as TV transmitters, computers, mobile phones, etc. This means that a unit may work perfectly in one part of the room but not at all in another.

Antenna The antenna (2) should be straight and pointed directly down to achieve best coverage between Visit Smoke Alarms and receivers within the Visit System.

### Activation

Via Test Button (1).  
Via the built-in smoke detector.  
Via the built-in temperature sensor, if the temperature exceeds approx. 57°C.

### Output signals

Built-in sound signal: 85 dBA maximum at 3 metres.

### Additional information

For indoor use only  
Dimensions Ø x H 100 x 35 mm  
With battery: 110 g. Without battery: 70 g

Weight:

Colour:

White

0086  
Bellman & Symfon AB /  
Homewatch Ltd  
09  
0086 - CPR- 537081  
EN 14604: 2005  
BE1480  
Technical data: see  
KN500468 held by the  
manufacturer.

For use in all EU countries.

Hereby, Bellman & Symfon AB,  
declares that this smoke alarm  
transmitter is in compliance with the  
essential requirements and other  
relevant provisions of directives:  
RED Directive 2014/53/EU.  
For a complete declaration of  
performance, visit [www.bellman.com](http://www.bellman.com)

Correct use and disposal of batteries:

Replace only with the same or  
equivalent type recommended by the  
manufacturer. Please dispose of old,  
defective batteries in an environmen-  
tally friendly manner in accordance  
with the relevant legislation.

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TM and © 2017, Bellman & Symfon AB, alle Rechte vorbehalten. Bellman, Bellman & Symfon
**Bellman & Symfon Deutschland GmbH**
und das Firmenlogo sind eingetragene Markenzeichen der Bellman & Symfon AB.