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
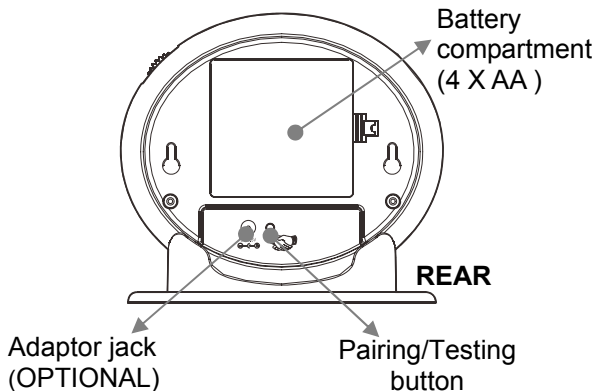
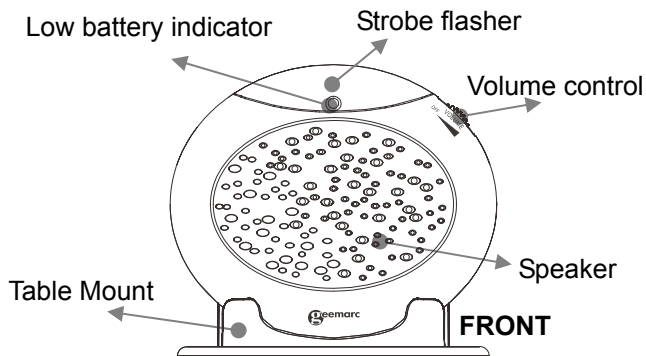
INSTALLATION

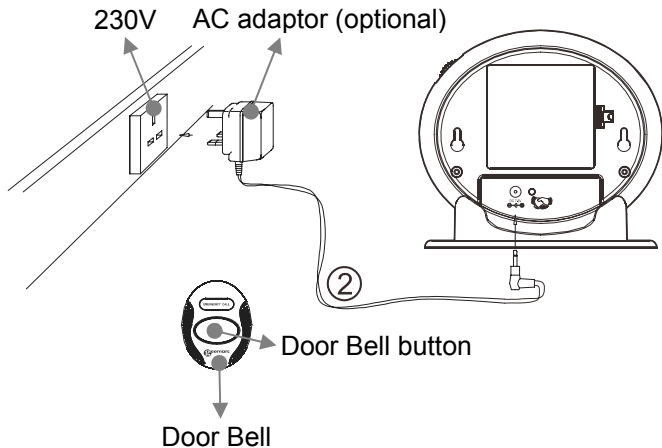Installation (See drawing)

1 – Insert the 4 (AA) batteries into the battery compartment (optional)

2 - Connect the adaptor 7.5V DC/1A (optional).

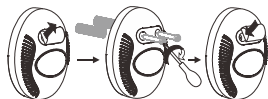
Electrical connection : The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as «hazardous voltage» according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the apparatus, ensure that the mains power socket is readily accessible.

INSTALLATION

Telephone connection : Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

Setting up the Door Bell

1. The door bell can be carried around by the people who need assistance
2. The door bell can be fixed on the wall or door surround. Drill two holes horizontally, 14mm apart. Insert the two supplied anchors into the two holes. Hang the door bell onto the two anchors and then insert and tighten the screws. Finally, stick the blank sticker over the two holes.

3. The door bell can be stuck onto a door or window using the supplied strong adhesive tape. Remove the protective paper from the adhesive tape on the back of the door bell and then stick to the required door, window or surface.

Wall Mounting

1. Drill two holes horizontally, 96mm apart, and leave the screws protruding from the wall by 5mm.
2. Hang the base on to the two screws and pull the base down to lock it on.

INSTALLATION

Table Mounting

If you want to mount the unit on the table simply place the unit into the base cradle, and then place the base cradle on the table.

Change the Battery of the Indoor Unit (optional)

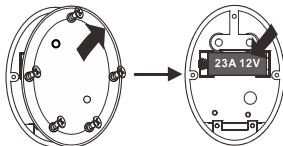
The unit requires 4×AA batteries

1. Open the battery door.
2. Insert four AA batteries into the compartment.
3. Snap the battery compartment door back into place.

Note: When the low battery indicator keeps flashing, it is time to replace the batteries. Alternatively, use a mains adaptor as an option (contact 01707 384483).

Change the Battery of the Door bell

The door bell is supplied with a 23A 12V battery. The door bell is supplied ready to use, it is not necessary to open the bell push when you first install the Amplicall 16. To replace the battery, unscrew the door bell from the wall, unscrew the five screws located on the base and open the box to get access to the battery compartment. Replace the battery, replace back and remount.

INSTALLATION

Note: Do not use a rechargeable battery. Do not put the used battery into the household rubbish, pay attention to battery regulations regarding battery disposal.

Volume Control Slide Switch

The volume control slide switch can be used to set the volume of the alarm sound and switch the power on/off.

The Pairing Procedure:

The supplied door bell has been paired with the base, there is no need to pair them. If the door bell is not paired with the base or you wish to pair the base with another door bell, please follow the pairing procedure below.

Press and hold the Pairing/Test button of the unit whilst pressing the doorbell button until the strobe flasher LED starts to flash. The unit will make a beep sound to confirm that pairing has taken place successfully.

To check whether the device is working properly and has been paired correctly, press the pairing/testing button once. The device should sound the alarm and flash.

Operation

Once the Amplicall 16 and the Door bell have been placed in the required locations and all required batteries have been inserted, you are ready to use the device. To use the device, the person who needs assistance must press the door bell button of the door bell.

TROUBLESHOOTING

Trouble shooting

Symptom	Solution
The unit makes the “dingdong” sound (ad hoc or continuously) without the door bell being pressed	1) Check the batteries are inserted correctly 2) Check the power adaptor is inserted into the unit and the wall correctly (if using mains power) 3) Turn off the power switch for 10 minutes and then turn it on
Low battery LED is flashing.	1) Turn off the power switch for 10 minutes and then turn it on 2) Insert new batteries into the base unit
Batteries and power adaptor are connected correctly, yet no “Dingdong” sound when the door bell is pressed.	1) Turn off the power switch for 10 minutes and then turn it on 2) Replace the base unit batteries 3) Replace the door bell batteries

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note : The guarantee applies to the United Kingdom only.

Declaration : Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

The declaration of conformity may be consulted at www.geemarc.com

GUARANTEE

Recycling directives : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

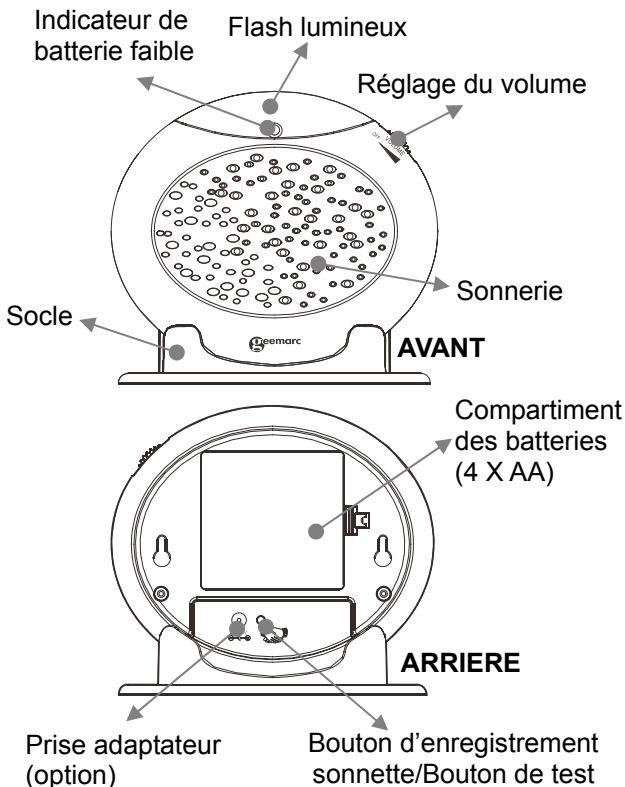
For product support and help visit our website
at www.geemarc.com

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