HUMANTECHNIK

***MF-1***

**D**

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Seite 2

Blitzmodul MF-1

**GB**

**Operating Instructions**

Page 6

Flash module MF-1

**F**

**Mode d'emploi**

Page 10

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**NL**

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Pagina 14

Flitsmodule MF-1

**I**

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Pagina 18

Modulo flash MF-1


**E**

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
Página 22

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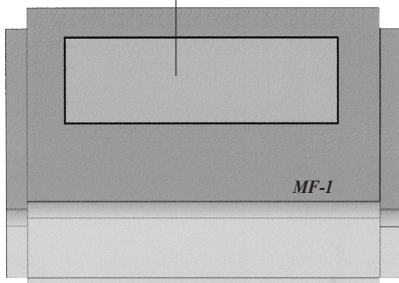Blitzfläche

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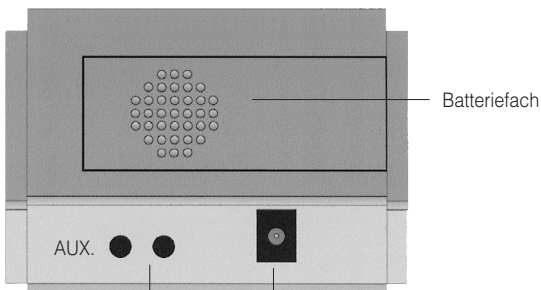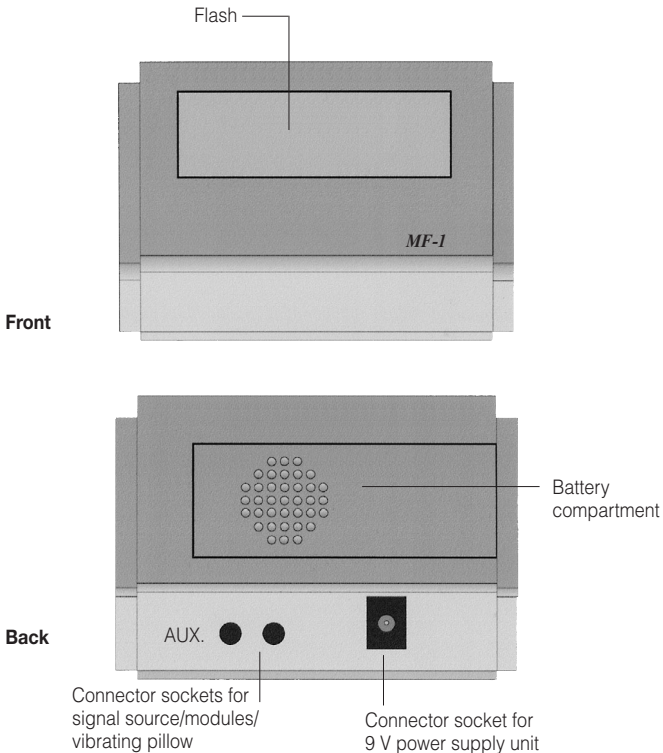
**The *flash module MF-1* can be connected to each of our units that has a connector socket (3.5 mm jack) for the vibrating pillow.**

We would like to congratulate you on the purchase of our *flash module MF-1* and hope that it will serve you well for a long time to come.

You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

With the *flash module MF-1* you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a flash signal.

### **Standard components**

Please check if all listed items are included:

- *Flash module MF-1*
- Connecting cable
- 9 V-power supply unit
- Operating instruction
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### **Installation**

Attach the connecting cable to one of the two connector sockets for the signal source at the back of the *flash module MF-1*. Plug the other end of the cable into a unit with a connector socket for a vibrating pillow e.g. alarm clock DS-1, VC-1/VC-10, time *flash/lisa* time *flash*, *lisa* receivers (table flash lamp, RF flash lamp, RF blink lamp, mobile RF receiver).

Plug the 9 V-power supply unit in the respective socket at the back of the *flash module MF-1*.

### Note

A vibrating pillow or another module (e.g. acoustic module) can be plugged into the empty connector socket at the back of the *flash module MF-1*.

### **Signalling**

Upcoming signals (e.g. wake-up alarm, *lisa* alarm) are signalled by very bright flashes.

### **Special features**

Power failure backup with a 9 V battery. (Battery not included)

The *flash module MF-1* can also be operated on just a 9 V battery

### **Maintenance and care**

The *flash module MF-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

### **Warranty**

The *flash module MF-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

### Technical data

Power supply: 9 V-power supply unit at 230 V, 50 H or with a 9 V battery

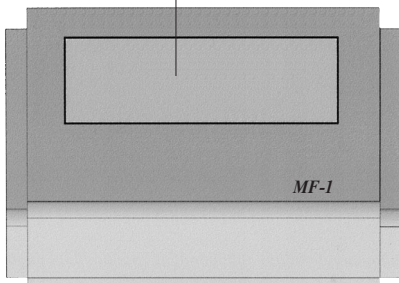
This product complies with European Union norms.


You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

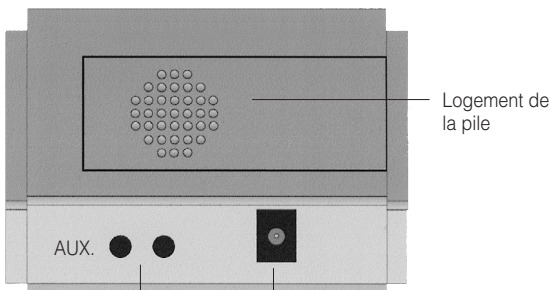Technical specifications subject to change without notice.

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**Humantechnik GHL AG**