RingFlash 250

Bedienungsanleitung (DE)
Mode d'emploi (FR)
Operating Instructions (UK)
Gebruiksaanwijzing (NL)
Manual de instrucciones (ES)

Istruzioni per l'uso (IT)
Bruksanvisning (NO)
Bruksanvisning (SE)
Käyttöohje (FI)
Manual de instruções (PT)

RingFlash 250

RingFlash 250

Bedienungsanleitung (DE)	3
Operating Instructions (UK)	18
Mode d'emploi (FR)	33
Gebruiksaanwijzing (NL)	48
Manual de instrucciones (ES)	63
Istruzioni per l'uso (IT)	78
Bruksanvisning (NO)	94
Bruksanvisning (SE)	109
Käyttöohje (FI)	123
Manual de instruções (PT)	138

Sicherheitsinformation

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Safety information

Please read this operating instruction manual thoroughly.

Under no circumstances complete any repair work on it yourself.

Intended use

The RingFlash 250 is a signal alarm enhancer intended for connection to telephones, doorbells and smoke alarms. Other uses are considered unintended use.

The RingFlash 250 is particularly suited for installation in loud environments and for people with hearing problems.

Unauthorised modification or reconstruction is not permitted.

Installation location

Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

Power supply

Caution: Only use the power adapter plug supplied because other power adapter plugs could damage the equipment. Ensure that access to the power adapter plug is not obstructed by furniture and such.

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre).

According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to both health and the environment!

Never open, damage or swallow batteries, nor allow them to get into the environment.

They may contain toxic and ecologically hazardous heavy metals.

You are legally obliged to dispose of batteries at the point of sale or at appropriate collection points provided by your local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that batteries must not be disposed of in normal domestic waste and but at authorised collection points.

Packaging materials must be disposed of according to local regulations.

Prolonged listening with the device at full volume could cause ear damage to the user.

RingFlash 250


Package contents

The package contains:


Content	RF250
RingFlash 250 base station	1
Power adapter plug (12VDC 1A)	1
Telephone connection cable	1
Pick-up	1
External microphone	1
12 Volt battery (23 A)	1
3 Volt battery (CR2)	1
Double-sided adhesive tape	2
Operating instructions	1

Operating Elements

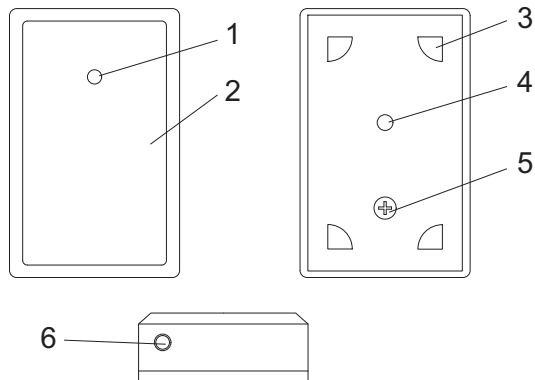
Pick-up without holder

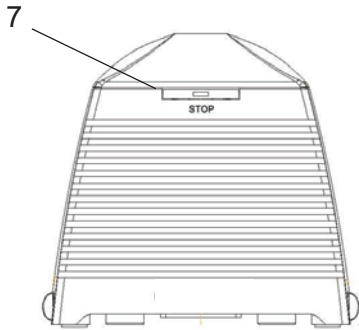
1. Status LED
2. Housing
3. Locating points for mounting bracket
4. Internal microphone
5. Housing screw
6. Jack socket for external microphone


Front view

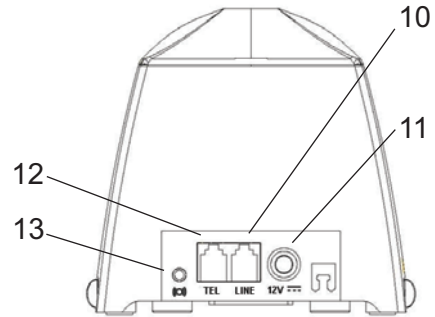Rear view

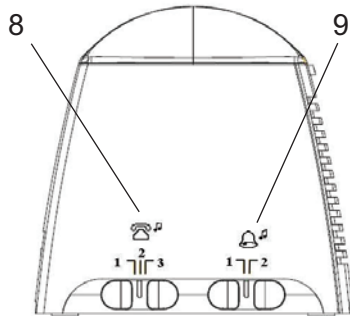Left side

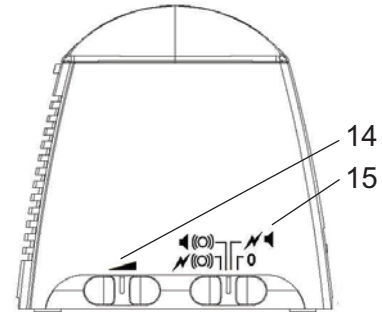
Right side

Using the RingFlash 250

- STOP button with LED
- Sliding switch for melody selection for the signal alarm in respect of the connected telephone
- Sliding switch for melody selection for the signal alarm in respect of the connected doorbell pick-up
- Telephone line connection port
- Power adapter plug connection port
- Vibration pad connection port (option)
- Sliding control for the signal alarm volume
- Sliding switch to select type of signal alarm

Connecting the base station

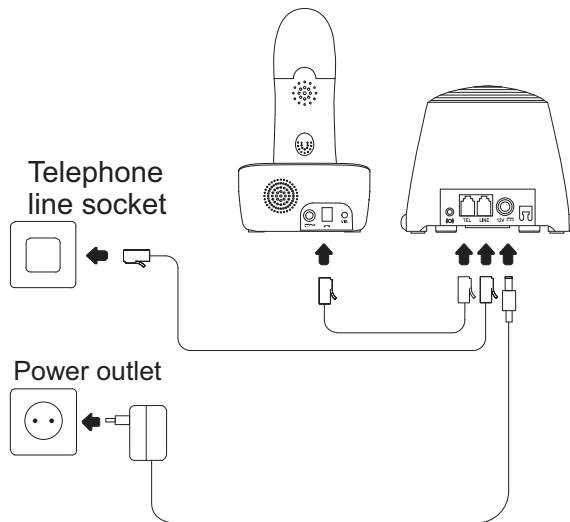Connect the RingFlash 250 base station as illustrated in the diagram. For reasons of safety, only use the power adapter plug and telephone connection cable contained in the material supplied.

 In some cases where the amplicomms Ring Flash is used in conjunction with older phones or those phones that do not comply to European Standard ES 203021 connectivity, either the RingFlash or telephone may not function correctly. If this happens you may notice the Ring Flash does not flash or ring or it is not possible to hear dial tone in the receiver when the telephone handset is lifted. To resolve this please retain your existing telephone line cord with your phone and use a double adaptor in the telephone socket (not supplied). Plug the telephone into one side of the double adaptor then plug your Ring Flash into the other available socket on the adaptor.

1. Disconnect the existing telephone connection line from the phone currently used.
2. Connect the end of the telephone cable just disconnected in the telephone line port LINE (10) in the RingFlash 250 base station.

3. Then plug the telephone line supplied in your telephone.
4. Connect the other end of the telephone cable to the telephone connection port TEL (12) in the RingFlash 250 base station.
5. Now connect the power adapter plug supplied to the power outlet.
6. Plug the cylindrical plug on the power adapter plug in the connection port for the adapter plug (11) in the RingFlash 250 base station.

 As soon as the RingFlash 250 base station is connected to the power supply network, the RingFlash 250 base station issues an acoustic signal to indicate correct installation. If there is no acoustic signal, disconnect the power adapter plug from the power socket. Check the connections on the RingFlash 250 base station. Wait for 1 minute and plug the power adapter back in the power socket.

Using the RingFlash 250

Settings on the RingFlash 250 base station

After connecting the RingFlash 250 base station correctly, you can define certain settings for the signal alarm. These settings include the type of signal alarm, melody for the signal alarm and volume of the signal alarm.

1. To begin, move the volume slide control (14) to a low volume (slide the control to the left).
2. Initiate a test call in order to test the volume of the signal alarm and then set the volume to the level required.

If you have not connected a phone to the RingFlash 250 base station, do not complete the volume adjustment for the signal alarm until you have connected a pick-up to the RingFlash 250 base station.

3. Select the type of signal alarm required using the sliding switch (15).
4. By moving the slider from left to right, you can select from the following types of signal alarm: flashing light and vibration, acoustic signal and vibration, flashing light and acoustic signal without vibration, no signal enhancement. (Also refer to Page 29 Quick function overview)
5. The sliding switches for selecting the melodies (8+9) can be used to select one melody for the signal alarm of the pick-up (9) connected to the RingFlash 250 base station and one melody for the signal alarm of the telephone (8) connected to the RingFlash 250 base station. (Also refer to Page 29 Quick function overview.)

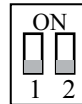
Preparing the pick-up

The pick-up can be used to detect the acoustic signals (rings or beeps) of the doorbell or smoke alarm currently used directly. The signals detected are then transmitted to the RingFlash 250 base station and issued as the type of signal alarm previously selected (special alarm signal for smoke alarm). Proceed as follows to install the pick-up:


1. Unscrew the housing screw (5) out of the cover of the pick-up.
2. Remove the loose, unscrewed cover from the housing (2) of the pick-up.

 You can switch the application for which the pick-up is used. You can use the pick-up for door bells or smoke alarms, for example. In order to be able to use both functions, you need an additional pick-up (option).


3. You can now adapt the settings according to your intended purpose. Change the setting of the 2-position mini DIP switch according to the following diagrams.

Doorbell 1

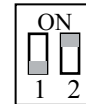Doorbell 2

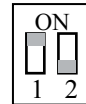Smoke alarm

 When set-up for doorbell 1, the LED in STOP button (7) lights up yellow; when set-up for doorbell 2, the LED in the STOP button (7) lights up green.

4. Insert the batteries (CR2 and 23A) in the holders provided. Pay attention that the batteries are inserted the right way around (correct polarity).
5. Place the housing cover of the pick-up back on the housing (2).

Using the RingFlash 250

6. Screw the housing screw (5) back in the housing (2) of the pick-up.
7. Insert the pick-up in the holder. Pay attention that the pick-up is fitted the right way around in the holder. The latches for the holder (3) on the pick-up help you to find the correct installation position.

Assembling on a door bell

1. Use the tape supplied to attach the holder of the pick-up on the doorbell. Position the holder so that the large opening in the holder is over the doorbell's sound generator.
2. Press your doorbell.
3. When the Status LED (1) on the pick-up lights up, the pick-up transmits a signal to the base station.

Assembling on a smoke alarm

1. Use the tape supplied to fix the pick-up holder beside the smoke alarm. Position the holder so that the pick-up is installed as close as possible to the sound generator of the smoke alarm.

 Never install the pick-up directly on the smoke alarm. If you position the pick-up directly on the smoke alarm, the function of the smoke alarm can no longer be guaranteed.

2. Activate the test function of your smoke alarm. (Refer to the operating manual supplied with the smoke alarm.)

 The pick-up must be exposed to a smoke alarm signal for 5 seconds. The transmission only triggers the RingFlash 250 base station after this period.

3. When the Status LED (1) on the pick-up lights up, the pick-up transmits a signal to the RingFlash 250 base station.
4. The RingFlash 250 base station continues to signal an alarm until the smoke alarm ceases to sound an alarm. (Refer to the operating manual supplied with the smoke alarm.)

 If the pick-up does not detect the acoustic signals, first reduce the distance between the pick-up and RingFlash 250 base station.

 If the acoustic signals cannot be picked up properly, it is possible that the door bell ringing volume is too low. Set the door bell to a louder volume or adjust the sensitivity of the integrated microphone from "Lo" to "Hi". The switch for this is located inside the pick-up. Open the pick-up as described in "Preparing the pick-up".

Connecting an external microphone

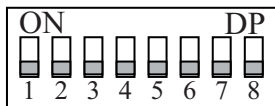
If the acoustic signals cannot be picked up properly by the internal microphone or the pick-up cannot be mounted on the device to be monitored, you can connect an external microphone.

1. Install the pick-up near the device to be monitored.
2. Use the tape supplied to fix the microphone to the loudspeaker to the device to be monitored.
3. Insert the connection plug from the external microphone in the jack socket (6) of the pick-up.

Interference through other equipment

Under certain circumstances, other electrical equipment can cause interference to the RingFlash 250. You can alter the access code of the RingFlash 250 and pick-up as follows:

1. Remove the small cover on the underside of the RingFlash 250 base station.
2. Change the setting of the 8-position DIP switch, illustrated below, as necessary.

3. Remove the cover of the pick-up housing as described on Page 25.
4. Change the setting of the 8-position mini DIP switch according to the settings programmed on the RingFlash 250 base station.
5. Replace the cover of the pick-up housing as described on Page 25.

Battery charge

The Status LED (1) of the respective pick-up flashes when the batteries reach a critical charge level. If the Status LED (1) flashes once every 5 seconds, change the 23A battery. If the Status LED (1) flashes twice every 5 seconds, change the CR2 battery.

Vibration alarm

You can insert a vibration pad (not contained in the material supplied) in the connection socket (13) on the rear side of the RingFlash 250 base station. If a signal is received, the vibration pad begins to shake.

Wall fixation

You can use suitable screws and wall dowels (not contained in the material supplied) to mount the RingFlash 250 base station, via the holes in its base, on a wall.

Setting the ring melody

There are 3 melodies available for selection on the RingFlash 250 base station. Simply move the sliding switch (8) on the left-hand side to select melody 1, 2 or 3.

Setting the doorbell melody

There are two melodies available for selection by moving the sliding switch (9) on the left-hand side to select melody 1 or 2.

Setting the alarm mode

There are 4 alarm modes available for selection. Use the sliding switch (15) on the right-hand side to select one of the following alarm modes.

Mode	Telephone	Doorbell
0	Off	Melody from loudspeaker
	Melody from loudspeaker and flashing light	Melody from loudspeaker and flashing light
	Melody from loudspeaker and external vibration	Melody from loudspeaker and external vibration
	Melody from loudspeaker, flashing light and external vibration	Melody from loudspeaker, flashing light and external vibration

Suppressing the ringing tone

Press the STOP button (7) to suppress the ringing tone for incoming calls.

Maintenance/Guarantee

Maintenance

- Clean the surface of the product using a soft, lint-free cloth.
- Never use cleaning agents or solvents.

Guarantee

Amplicomms equipment is manufactured and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of guarantee do not apply where the cause of equipment malfunction is the fault of the telephone network operator or any interposed private branch extension systems. The terms of guarantee do not apply to the batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase. All deficiencies related to material or manufacturing errors within the period of guaran-

tee will be redressed free of charge. Rights to claims under the terms of guarantee are annulled following tampering by the purchaser or third parties. Damage caused as the result of improper handling or operation, normal wear and tear, incorrect positioning or storage, improper connection or installation, Acts of God and other external influences are excluded from the terms of guarantee. In the case of complaints, we reserve the right to repair defective parts, replace them or replace the entire device. Replaced parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your device shows signs of a defect within the period of guarantee, please contact the sales outlet where you purchased the Amplicomms device, producing the purchase receipt as evidence. All claims under the terms of guarantee in accordance with this agree-

ment can only be asserted at the sales outlet. No claims under the terms of guarantee can be asserted after a period of two years from the date of purchase and hand-over of the product.

Declaration of Conformity

This equipment fulfils the requirements stipulated in the EU Directive:

1999/5 EC directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the above mentioned directive is confirmed by the CE Mark on the device. To view the complete Declaration of Conformity, please refer to the free download available at our web site www.amplicomms.com.

Audioline GmbH, D-41460 Neuss

Important Notes on the Guarantee

Important Notes on the Guarantee

The period of guarantee for the Amplicomms product begins on the date of purchase. Rights to claims under the terms of guarantee relate to deficiencies resulting from material or production faults. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, normal wear and tear, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

CLAIMS UNDER THE TERMS OF GUARANTEE MAY ONLY BE ASSERTED TO THE RETAILER WHERE THE EQUIPMENT WAS PURCHASED, WHEREBY THE PURCHASE RECEIPT MUST ALSO BE SUBMITTED.

TO PREVENT UNNECESSARY LONG DELAYS, PLEASE DO NOT SEND EQUIPMENT UNDER COMPLAINT TO OUR OFFICE ADDRESS.

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AUDIOLINE GmbH