# dry-go





Electronic mini drying station for hearing aids.

INSTRUCTION MANUAL

EN INSTRUCTION MANUAL  
DE BEDIENUNGSANLEITUNG  
FR MANUEL D'UTILISATION  
ES MANUAL DE INSTRUCCIONES  
PT MANUAL DE INSTRUÇÃO  
IT MANUALE D'USO  
NL HANDLEIDING  
TR MANUALE D'ISTRUZIONI  
DK BETJENINGSVEJLEDNING

FI KÄYTTÖOHJEET  
SE BRUKSANVISNING  
NO INSTRUKSJONSHÅNDBOK  
HU HASZNÁLATI UTASÍTÁS  
PL INSTRUKCJA OBSŁUGI  
CZ UŽIVATELSKÁ PŘÍRUČKA  
SL NAVODILA ZA UPORABO  
BG РЪКОВОДСТВО ЗА ЕКСПЛОАТАЦИЯ  
RU РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ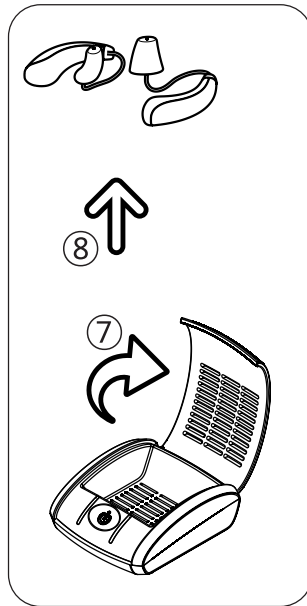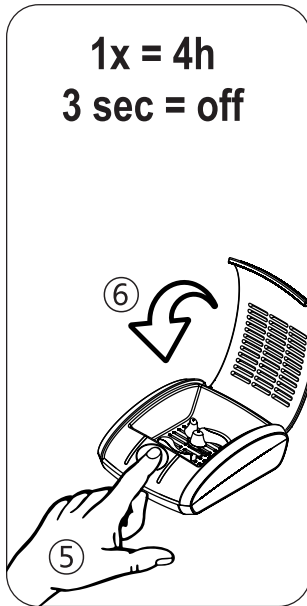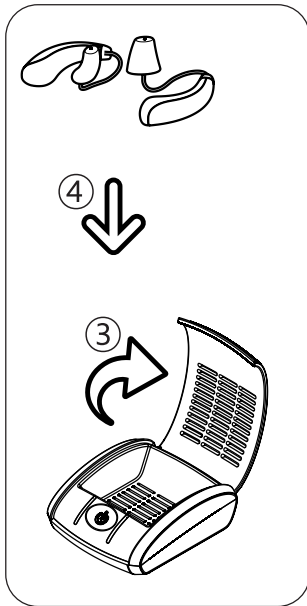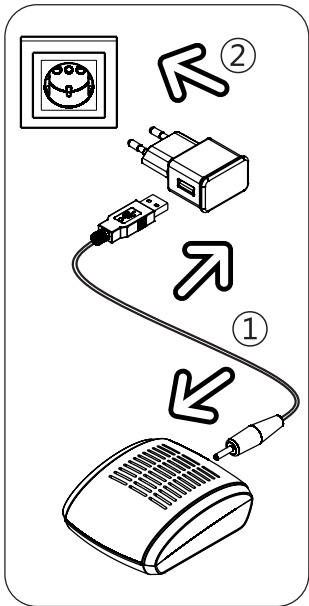

### **Dries and cares your hearing systems**

Moisture and ear wax can impair the function of hearing systems and cause failures.

The dry-go offers optimal care. Comfort drying at 50°C extends life time significantly.

### **Using first time**

1. Insert the USB-cable into the 5 V power supply and the round plug connector into the jack on the rear side of your drying station
2. Then plug the power supply into an AC power socket

### **Operation**

3. Open the lid of your dry-go
4. Place the with wet wipes pre-cleaned hearing systems with the battery-door open (if possible) into your drying station
5. Briefly touch the sensor button and the comfort drying program is starting automatically
  - The 4 hours drying program with approx. 50°C warm air is fully processor controlled
6. Close the lid
  - After 4 hours the drying station automatically shutdown to stand-by
7. Open the lid again
8. Remove the dried and hygienically cleaned hearing systems

① The dry-go is ready to use

① LED permanent light

① LED switch off

## Maintenance and operation

- We recommend wet wipes for cleaning your dry-go
- You can interrupt the drying process by holding the sensor button for more than 3 seconds
- The dry-go is very energy efficient; unplug the power supply only for prolonged absence

## Warnings

- Avoid contact with water
- Don't use the dry-go near basins etc.
- In the event of a malfunction, unplug the power supply and take the drying station to your retailer
- Don't disassemble the drying station and use only the supplied 5 V power adapter and USB-cable
- Children must be supervised to ensure that they don't use the drying station as a toy
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station
- The drying station has an automatic room and temperature modulation as well as safety shutdowns

- Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities
- Contact your local government for information regarding the collection systems available
- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being

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